devices such as pacemakers and hearing aids. If you use such devices, consult your physician before using this product. Do not use or store your device in areas with high concentrations of moisture or humidity, such as showers, saunas, swimming pools, or near large bodies of water. Do not use the device in malfunction. On aircraft, ships and other means of transport, the use of electronic devices could interfere with electronic navigational instruments. Comply with the provisions of the transport service provider. Turn off the device in potentially hazardous situations, such as gas stations or near fuel. Do not store your device near magnetic fields. Not hold the screen too close to your eyes. If you experience discomfort, such as muscle spasms, or become disoriented, stop using the device immediately and consult your physician. To avoid a short circuit, frequent breaks when using the device. Damage or poor performance caused by failure to follow the above warnings and instructions may void the manufacturer's warranty. Epsinret is not responsible for performance problems or incompatibilities caused by unauthorized modification of system settings or software.

System and hardware requirement:
Android 5.0 or later / iOS 10.0 or later

PLEASE SCAN THE CODE BELOW TO ACCESS THE COMPLETE MANUAL:

English – EN
Please read this manual and pay attention to the safety warnings before using the product
Keep the manual to have the possibility of consulting it at a later date and in the event of passing the product on to a third party. Epsinret S.p.A. cannot in any way be held responsible for injury or damage caused to people or things deriving from improper use of the product.

The results of the functions offered are for reference only, can not be used as a definitive condition of the human body, can not be used for medical testing purposes, not as a body health judgment.

System and hardware requirement:
Android 5.0 or later / iOS 10.0 or later
PLEASE SCAN THE CODE BELOW TO ACCESS THE COMPLETE MANUAL:

Preparation:
The product must be activated by charging it fully before first use and hold the right power key. After that, make sure you keep your device charged enough.

Clip the connector to the product body and insert into the USB port of a charger to activate it (pay attention to insert it correctly and use the correct current).

Verify that your phone meets the requirements.

Download the APP to connect.

Please search "DA FIT" APP in the App Store or Android store to scan the QR CODE

Association:
Make sure your smartphone's wireless functionality is turned on and open the Smart-Time Pro app.

When you analyze your device. Right-click the product to wake up the device during the process and complete the pairing.

Usage:
Once you connect to the app, it will synchronize the device and analyze your activities, oxygen saturation, heart rate and blood pressure.

Touch key - Press and hold the touch key: switch interface - Single touch of the touch key: enter
Bracket functions: Time + Pedometer mode + Trip distance mode + Calories mode + Heart rate mode + Blood pressure mode + Oxygen saturation mode + Music management + Messages + Answer call

The device is waterproof but not suitable for immersion in water.

1. Time mode: display time, battery, date, Press and hold the touch key to switch between available interfaces

2. Pedometer: Automatically record the number of steps

3. Distance traveled: automatically records the distance traveled

4. Calories: automatically calculates the calories burned

5. Heart Rate Monitor: The device will measure your heart rate automatically. You can also manually measure your heart rate via APP. The page will display the most recent measurement data.

6. Blood pressure monitor: The device will measure your current blood pressure situation. The page will show the latest measurement data (measurement time is about 30 seconds).

7. Oxygen Saturation Monitor: The device will measure your blood oxygen saturation levels. You can also manually measure oxygen saturation via APP. The page will display the most recent measurement data.

8. Sleep Monitor: Automatically recognize your status and track your entire sleep progress with analysis of deep sleep and light sleep hours.

9. Notifications: the bracelet will vibrate when the phone rings, you can answer directly from the smartwatch thanks to the built-in microphone and speaker

10. Message notifications: the bracelet will vibrate when the phone receives a message

11. Weather forecast: thanks to the included GPS function, the product is able to promptly provide the weather conditions of the place where you are

12. Music control: Once you activate music on your device, you can control and play songs from your smartwatch

13. Breath: Thanks to the "breath" function, you can control your breathing for a second/5 minutes, following the instructions on your smartwatch

Questions and answers:

(1) Make sure the wireless function is turned on.

(2) Make sure the distance between your phone and the device is not more than 0.5m.

(3) If the device is not charged, it will not be able to connect fully charged, please contact us.

2. Can't connect the device with the app?

Restart your wireless or smartphone for device pairing

3. The APP shows no data? Make sure the battery is charged

4. Wrong time: If the bracelet is low power, the time may be wrong. Please charge the bracelet and connect the phone again.

If you wear a pacemaker or other implanted electronic device, please consult your doctor. If you can wear this product, the device will not harm your health.

If you wear a heart rate, blood pressure and oxygen saturation device, please consult your doctor. Wear the device for a long time, please contact us.

5. Notifications: the bracelet will vibrate when the phone receives a message

6. Can't connect the device with the app?

Restart your wireless or smartphone for device pairing

3. The APP shows no data? Make sure the battery is charged

4. Wrong time: If the bracelet is low power, the time may be wrong. Please charge the bracelet and connect the phone again.

If you wear a pacemaker or other implanted electronic device, please consult your doctor. If you can wear this product, the device will not harm your health.

If you wear a heart rate, blood pressure and oxygen saturation device, please consult your doctor. Wear the device for a long time, please contact us.

5. Notifications: the bracelet will vibrate when the phone receives a message

6. Can't connect the device with the app?

Restart your wireless or smartphone for device pairing

3. The APP shows no data? Make sure the battery is charged

4. Wrong time: If the bracelet is low power, the time may be wrong. Please charge the bracelet and connect the phone again.

If you wear a pacemaker or other implanted electronic device, please consult your doctor. If you can wear this product, the device will not harm your health.

If you wear a heart rate, blood pressure and oxygen saturation device, please consult your doctor. Wear the device for a long time, please contact us.

5. Notifications: the bracelet will vibrate when the phone receives a message

6. Can't connect the device with the app?

Restart your wireless or smartphone for device pairing

3. The APP shows no data? Make sure the battery is charged

4. Wrong time: If the bracelet is low power, the time may be wrong. Please charge the bracelet and connect the phone again.

If you wear a pacemaker or other implanted electronic device, please consult your doctor. If you can wear this product, the device will not harm your health.

If you wear a heart rate, blood pressure and oxygen saturation device, please consult your doctor. Wear the device for a long time, please contact us.

5. Notifications: the bracelet will vibrate when the phone receives a message

6. Can't connect the device with the app?

Restart your wireless or smartphone for device pairing

3. The APP shows no data? Make sure the battery is charged

4. Wrong time: If the bracelet is low power, the time may be wrong. Please charge the bracelet and connect the phone again.

If you wear a pacemaker or other implanted electronic device, please consult your doctor. If you can wear this product, the device will not harm your health.

If you wear a heart rate, blood pressure and oxygen saturation device, please consult your doctor. Wear the device for a long time, please contact us.

5. Notifications: the bracelet will vibrate when the phone receives a message

6. Can't connect the device with the app?

Restart your wireless or smartphone for device pairing

3. The APP shows no data? Make sure the battery is charged

4. Wrong time: If the bracelet is low power, the time may be wrong. Please charge the bracelet and connect the phone again.

If you wear a pacemaker or other implanted electronic device, please consult your doctor. If you can wear this product, the device will not harm your health.

If you wear a heart rate, blood pressure and oxygen saturation device, please consult your doctor. Wear the device for a long time, please contact us.

5. Notifications: the bracelet will vibrate when the phone receives a message

6. Can't connect the device with the app?

Restart your wireless or smartphone for device pairing

3. The APP shows no data? Make sure the battery is charged

4. Wrong time: If the bracelet is low power, the time may be wrong. Please charge the bracelet and connect the phone again.

If you wear a pacemaker or other implanted electronic device, please consult your doctor. If you can wear this product, the device will not harm your health.

If you wear a heart rate, blood pressure and oxygen saturation device, please consult your doctor. Wear the device for a long time, please contact us.

5. Notifications: the bracelet will vibrate when the phone receives a message

6. Can't connect the device with the app?

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