A B C
User manual

EMERGENCY SLEEPING BAG

nexa

EN

L - Life-threatening situation

Check the scene and form an initial impression. Assess the situation and check for any potential hazards. If the situation is not safe and you cannot help without putting yourself in danger, call 112. Then wait for emergency personnel.

AIRWAY

Check the airway. If the injured individual is unresponsive, tilt the head back and lift the chin to open the airway.

- Unconscious – breathing normally; place in a stable lateral position. Continue to step B; bleeding.
- Unconscious – not breathing; call 112 and start cardiopulmonary resuscitation, CPR.

BLEEDING

- Stop external bleeding by applying pressure to the wound.
- Signs of internal bleeding; pain, pale and feels cold, rapid and shallow breathing.

CIRCULATION

If signs of shock, and in all major injuries, circulatory failure must be prevented.

- Do this:
- Allow the person to lie down and try to keep the injured individual calm.
 - Prevent hypothermia by placing blankets, sweaters, or jackets on and under the injured individual.

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Emergency sleeping bag

Intended use:

Emergency sleeping bag for thermal protection against heat, cold, and humidity in emergency situations.

Instructions for use:

Two-sided bag with different functions:

- Orange side outward to retain heat—helps maintain body temperature and shields from wind and humidity.
- Silver side outward to reflect heat, use to stay cool.

Warnings:

Do not cover your head, risk of suffocation. Single use only. Do not use during storms, near open flames/hot surfaces, or when electrosurgical devices are in use. Do not cut or puncture. Store in a dry place.

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

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